


Start Up:

- This product has two power sources: external power adaptor and internal battery.
 - Insert the power adaptor horizontally into the power port
- Interlock connection**
 - Align the red dots, or otherwise the interlock will be damaged
- Optical fiber and laser aperture connection**
 - Remove the dust cover and insert the optical fiber horizontally into the laser aperture
 - Do tighten it up, otherwise the device will not function
- Finger switch and finger switch interface connection**
 - Align the red dot on the finger switch with that on the finger switch port, and then insert it into the interface.
- Toggle the Power On/Off switch
- Enter Passcode: 0000

Start an ADT Treatment:

- Press **RING** to select Species
- Select Category > Condition > Body Part > Coat Color > Chronicity > Weight
- Put on protective goggles
- Select **Lens**
- Press **START**
- Press **Finger Switch** to activate treatment

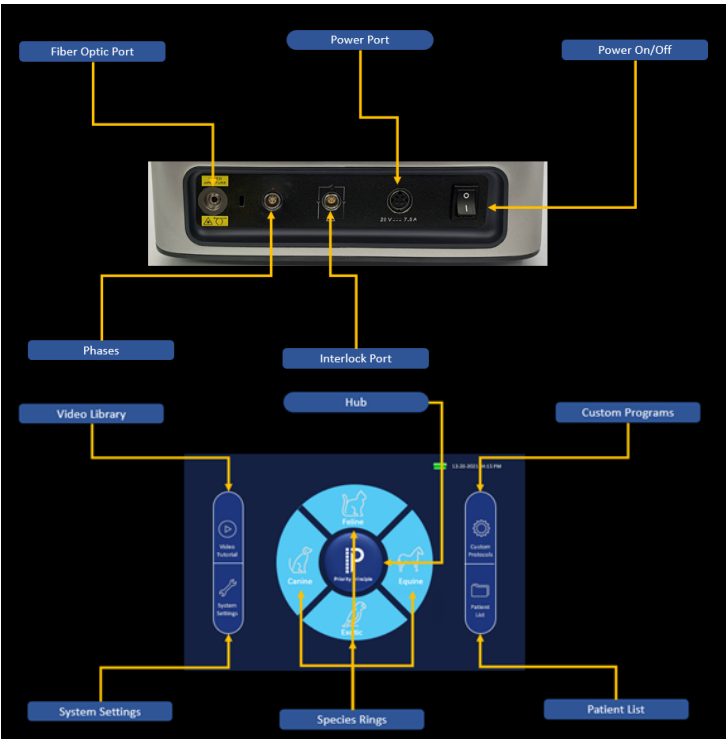

Navigation:

- Center Hub:
 - Species Rings**
 - Hub: Priority Principle**
- Left Bank:
 - Video Library**
 - System Settings**
- Right Bank
 - Custom Protocols**
 - Up to 100 Protocols
 - Quick Start Saved Protocols
 - Does not utilize Smart Dosing
 - Patient List**
 - Up to 3000 Patient Files
 - Searchable by Species, Patient Name, Owner or Contact Number

ADT Protocols	Priority Principle	Manual
ADT will modify dose (Power and Time) based on clinical input	10 Presets based on the Priority Principle Symptom based Simplest and easiest way to get results	Allows providers to combine any of the 4 wavelengths to create unique and custom protocols based on clinical studies and effectiveness.
Category → Condition → Body Part → Coat Color → Chronicity → Weight	Dose is modified by animal	Can be run entirely in Continuous Wave (CW) or pulsed modes with the widest range of pulse parameters (from 0 Hz or CW to 20,000 Hz).
Canine, Feline, Equine, Exotic	Canine, Feline, Equine, Exotic	Custom

Start Priority Principle Treatment:

- Activate *Priority Principle* by pressing the center **HUB**
- Press **RING** to select *Species*
- Select the **Priority**
- Put on protective goggles
- Select **Lens**
- Press **START**
- Press **Finger Switch** to activate treatment

Zoom	Massage	Massage Plus	Plus
3 – 4 cm ² Non-Contact Scanning Small Animal	4 cm ² Contact Point – Point Small Animal	20 cm ² Contact Point – Point Large Animal	20 cm ² Non-Contact Scanning Large Animal